

METHOD OF USE FOR REGAN-LOWE DEEP

1. Bring medium to room temperature prior to inoculation.
2. Use Calcium alginate or Dacron swab (uro-genital swabs work well) to collect specimen from nasopharynx.
3. Submerge tip of swab well into medium.
4. Break or cut any portion of the swab that is protruding from the tube.
5. Tighten cap.
6. Transport to laboratory within 24 hours at room temperature.

REFERENCE

Hardy diagnostics HUGO, 1999

7/2/08
11/6/00
12/8/05
11/30/06
12/31/07