

VALLEYCARE
OLIVE VIEW-UCLA MEDICAL CENTER/HEALTH CENTERS
DEPARTMENT: FACILITIES
POLICY & PROCEDURE

NUMBER: 580
VERSION: 1

SUBJECT/TITLE: ULTRASONIC CLEANER

POLICY: Service every six (6) months.
PURPOSE:

PROCEDURE:

1. Check condition of power cord and plug.
2. Remove all deposits from tank.
3. Check contact or transducer with tank body.
4. Verify integrity of electrical wiring and mechanical components.
5. Check operation of unit.
6. Check sealing of tank cover.
7. Clean unit as necessary.
8. Check tank for leaks or crystallization.
9. Check electrical leakage current and grounding integrity.

POLICY: Service every twelve (12) months.

PROCEDURE:

10. Clean and lubricate cooling fan as required.
11. Clean or change filters as applicable.
12. Tune oscillator using manufacturer's instructions.

DEPARTMENTS: FACILITIES

DEFINITIONS:

SUBJECT/TITLE: ULTRASONIC CLEANER

Policy Number: 580

Page Number: 2

References:	
Approved by: Robert Ross (Director, Facilities)	Date: 08/12/2009
Review Date: 07/12/2015	Revision Date:
Distribution: Facilities	
Original Date: 08/12/2009	