

VALLEYCARE
OLIVE VIEW-UCLA MEDICAL CENTER/HEALTH CENTERS
DEPARTMENT: FACILITIES
POLICY & PROCEDURE

NUMBER: 616
VERSION: 1

SUBJECT/TITLE: SLIDE STAINER

POLICY: Service every six (6) months.

PURPOSE:

- PROCEDURE:**
1. Check physical condition of power cord and plug.
 2. Check mechanical integrity of switches, controls, indicators, etc.
 3. Clean interior of unit including platen and conveyor spirals.
 4. Check all tubing for occlusion and leakage.
 5. Check the peristaltic pumps for proper operation. Lubricate if necessary.
 6. Lubricate the blower bearings and the spiral feed bar bearings.
 7. Inspect gears and micro switches for possible defects.
 8. Perform operational check and adjust as necessary.
 9. Check electrical leakage current and grounding integrity.

DEPARTMENTS: FACILITIES

DEFINITIONS:

References:	
Approved by: Robert Ross (Director, Facilities)	Date: 08/12/2009
Review Date: 07/12/2015	Revision Date:
Distribution: Facilities	
Original Date: 08/12/2009	