

CHECKING THE ZOLL AUTOMATED EXTERNAL DEFIBRILLATOR (AED) PLUS

The Zoll AED Plus will be checked on the first Monday of each month and recorded on the AED Inspection Tag.

Three main things to check when inspecting the AED:

1). Verify that the green check mark is displayed in the status window.

If a red "X" is displayed, follow the troubleshooting instructions located in the Zoll AED Plus Operators' Manual.

2). Check the expiration date of the AED pads (Adult and Pediatric)

Most Zoll AEDs utilize the CPR-D Pads. If the pads are beyond the expiration date, they should be replaced.

3). Check the overall condition of the AED

Check for signs of damage (case, cables, wires). Check Battery is within the expiration date. Batteries installed in the Zoll AED Plus typically have a 5 year life from the date of installation.

The Operator's Manual is located on the Olive View-UCLA Intranet under Training Resources → Zoll Defibrillator Tutorial → Zoll AED Plus Operators' Guide.